


HOOF-it II Bovine Blocking Directions:



Trim the affected claw.

Treat the lesion.

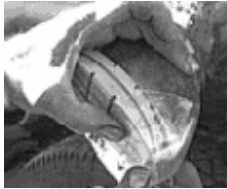

Select the right or left Hoof-it® block and boot (the block will be applied to the healthy claw).

In the boot, mix the liquid and powder to form a composite with the consistency of rubber-glu.



Insert the wood block into the boot, pressing the block into the acrylic mixture.

Spread the extra acrylic thickly onto the wood block.

Attach the *Hoof-it* boot and block to the claw.



Spread the remaining composite over and on the sides of the boot and claw.

Allow the acrylic to set (2-3 minutes).

Release the cow from the chute

